For professional in vitro diagnostic use only.

INTENDED USE

Vitassay Crypto+Giardia+Entamoeba is a rapid, immunochromatographic assay for the simultaneous qualitative detection of *Cryptosporidium*, *Giardia*, *Entamoeba* (*histolytica* and *dispar*) in human stool samples.

Simple, non-invasive and highly sensitivity immunoassay to make a presumptive diagnosis of *cryptosporidium, giardia* and/or *entamoeba* infection.

INTRODUCTION

Although there could be many other causes of diarrea, the enteric protozoa *E. histolytica, G. lamblia* and *Cryptosporidium spp.* have been recognized as important causes of diarrhea among human beings. *E. histolytica* is a pathogenic parasite for which humans are the primary reservoir. The clinical presentation can range from asymptomatic carriage to gastrointestinal disease and invasive disease. *E. histolytica* is morphologically identical to the nonpathogenic species *E. dispar* and *E. moshkovskii*, though genetic differences have confirmed their separation into independent species.

G. lamblia (synonyms: *G. intestinalis* and *G. duodenalis*) is the most common protozoan infection of the intestinal tract worldwide. Many countries, especially developing countries, show a high infection rate of giardiasis. It is believed that giardiasis is still a significant health problem. Most infected persons are children who suffer and experience growth retardation. Cryptosporidiosis is a frequent cause of diarrheal disease in humans. Infection is acquired via feco-oral route, and *C. parvum* has been recognized as the cause of large waterborne and food-borne outbreaks of gastroenteritis. Patients tend to present with a self-limiting diarrhea that may last for several weeks to months.

Transmission of intestinal parasites depends on the presence of infected individuals, poor sanitation and principally, the socioeconomic and behavioral factors in the population.

In addition, giardiasis, cryptosporidiosis and amoebiasis may result on severe diarrhea, impared intestinal absorptive function and malnutrition, conditions often associated with poor cognitive function and failure to thrive in early childhood.

All these three parasites can cause waterborne outbreaks, and also foodborne outbreaks have been reported.

PRINCIPLE

Vitassay Crypto+Giardia+Entamoeba is a qualitative immunochromatographic assay to make a presumptive diagnosis of *Cryptosporidium, Giardia, Entamoeba (histolytica and dispar)* infection.



Strip A: The test line zone of the nitrocellulose membrane is precoated with monoclonal antibodies against *Crypto*.

Strip B: The test line zone of the nitrocellulose membrane is precoated with monoclonal antibodies against *Giardia*.

Strip C: The test line zone of the nitrocellulose membrane is precoated with monoclonal antibodies against *Entamoeba (histolytica and dispar).*

During the process, the sample reacts with the antibodies against Crypto (strip A) and/or Giardia (strip B) and/or Entamoeba (strip C), forming conjugates. The mixture moves upward on the membrane by capillary action. If the sample is Crypto positive, antibodies present on the membrane (test line) capture the conjugate complex and a red line will be visible in the strip A, if the sample is *Giardia* positive, antibodies present on the membrane (test line) capture the conjugate complex and a red line will be visible in the strip A, if the sample is *Giardia* positive, antibodies present on the membrane (test line) capture the conjugate complex and a red line will be visible in strip B and if the sample is *Entamoeba* positive, antibodies present on the membrane (test line) capture the conjugate complex and a red line will be visible in strip C. Although the sample is positive or negative, the mixture continues to move across the membranes and the green control line always appears (for all the strips).

The presence of a green line (in the control zone (C)) indicates that sufficient volume is added; proper flow is obtained and serves as an internal control for the reagents.

PRECAUTIONS

- For professional in vitro use only.
- Do not use after expiration date.
- Do not use the test if its pouch is damaged.
- Specimens should be considered as potentially hazardous and handle in the same manner as an infectious agent. A new test must be used for each sample to avoid contaminations errors. Single use device.
- Tests should be discarded in a proper biohazard container after testing.
- Reagents contain preservatives. Avoid any contact with the skin or mucous membrane. Consult safety data sheet, available on request.
- Components provided in the kit are approved for use with the Vitassay Crypto+Giardia+Entamoeba. Do not use any other commercial kit component.
- Follow Good Laboratory Practices, wear protective clothing, use disposal gloves, goggles and mask. Do not eat, drink or smoke in the working area.

STORAGE AND STABILITY

Store as packaged in the sealed pouch either at refrigerated or room temperature (2-30°C/35.6-86°F).

VITASSAY

Crypto+Giardia+ Entamoeba

Rapid test for the simultaneous qualitative detection of cryptosporidium, giardia, entamoeba (histolytica and dispar) in human stool samples.

IUE-7715037 Ed01 April 2017





The test is stable until the expiration date printed on the sealed pouch.

The test must remain in the sealed pouch until use. Do not freeze.

MATERIALS

MATERIAL PROVIDED	MATERIAL REQUIRED BUT NOT PROVIDED
 10 tests/kit Vitassay Crypto+Giardia+Entamoeba Instructions for use. 10 vials with diluent for sample dilution. 	 Specimen collection container. Disposable gloves. Timer. Spatula.

SPECIMEN COLLECTION

Collect sufficient quantity of feces: 1-2 g or mL for liquid samples. Stool samples should be collected in clean and dry containers.

The samples can be stored in the refrigerator $(2-8^{\circ}C/36-46.4^{\circ}C)$ for 1-2 days prior to testing. For longer storage, maximum 1 year, the specimen must be kept frozen at $-20^{\circ}C/-4^{\circ}F$. The samples will be brought to room temperature before to testing.

Homogenise stool sample as thoroughly as possible prior to preparation.

SPECIMEN PREPARATION

- 1 Remove the cap of the vial with diluent for sample dilution (figure 1) and use the spoon to collect sufficient sample quantity. For solid stool, insert the spoon in 4 different areas of the stool sample (figure 2), remove any excess sample with a spatula (figure 3), and place the spoon cap back into the vial for sample dilution (figure 4). For liquid stool, take a spoonful of the sample (figure 3) and transfer it into the vial for sample dilution.
- 2.Close the vial for sample dilution tightly and shake it to dilute and mix the sample with the diluent (figure 4).





.

Allow the test, stool sample, controls and diluent to reach room temperature ($15-30^{\circ}C/59-86^{\circ}F$) prior to testing. Do not open pouches until the performance of the assay.

1. Shake the vial with the sample vigorously to obtain a good sample dilution.

2. Remove the **Vitassay Crypto+Giardia+Entamoeba** from its sealed bag just before using it (figure 5).

3. Take the vial for sample dilution containing the diluted sample (figure 6), place it inside the multiplex tube (figure 7). Screw the cap of the multiplex tube tightly (figure 8). The bottom of the vial



for sample dilution will break and the diluent+sample solution reaches the sample zone of the strips (figure 9).

4. Leave the multiplex tube vertically on a flat surface and read the results at **10 minutes**. Do not read the test results later than 10 minutes.

If the test does not run due to solid particles (the sample is not homogenized), migration process can stop on one or more strips. In this case, tap the end of the multiplex tube on hard surface to allow migration to start again.



F09-06 Rev00



Reaction takes place. Read results at 10 minutes.

Close the cap and

the bottom of the

diluent vial will

break.

INTERPRETATION OF THE RESUETS

Strip A: Crypto, Strip B:Giardia and Strip C: Entamoeba

	NEGATIVE	
C T annolea landa Cyppo Cyppo A B C	Only one green line in the control zone (C) in the three strips (A,B and C)	There is no Cryptosporidium, Giardia and Entamoeba (histolytica o dispar) presence.
Ent	POSITIVE	
amoeba amoeba iardia iardia iardia iardia		





Notes: The intensity of the red coloured test line in the result line region (T) will vary depending on the concentration of antigens in the specimen.

Positive results detailed in the above table should be followed up with additional confirmatory diagnostic procedures.

Single or dual simultaneous infections are more frequent than triple.

Invalid results: Total absence of any control coloured lines (green) indicates an invalid result, regardless of the appearance or not of the test lines (red). Wrong procedural techniques or deterioration of the reagents are mostly the main reasons for control line failure. Review the procedure and repeat the assay with a new test. If the problem persists, discontinue using the kit and contact your local distributor.

QUALITY CONTROL

Internal procedural controls are included in **Vitassay Crypto+Giardia+Entamoeba**. Green lines appearing in the in the results window are internal controls, which confirm sufficient specimen volume and correct procedural technique.

LIMITATIONS

- Vitassay Crypto+Giardia+Entamoeba must be carried out within 2 hours of opening the sealed bag.
- An excess of stool sample could cause wrong results (brown bands appear). Dilute the sample with the diluent and repeat the test.
- Only fresh or fresh frozen unpreserved and unfixed stool samples can be tested.
- The intensity of test line may vary depending on the concentration of antigens.
- The use of other samples different from human fecal samples has not been established.



- •The quality of Vitassay Crypto+Giardia+Entamoeba depends on the quality of the sample; Proper faecal specimens must be obtained.
- After one week of infection, the number of parasites in feces is decreasing, making the sample less reactive. Stool samples should be collected within one week of the onset symptoms.
- Positive results determine the presence of cryptosporidium and/or giardia and/or entamoeba in fecal samples. A positive result should be followed up with additional laboratory techniques (biochemical methods, PCR, microscopy) to confirm the results. A confirmed infection should only be made by a physician after the evaluation of all clinical and laboratory findings and must be based in the correlation of the results with further clinical observations.
- •Negative results should not be considered as conclusive; it is possible that the concentration of antigen is lower than the detection limit value. If symptoms or situation still persist a cryptosporidium and/or giardia and/or entamoeba determination should be carried out with another technique (for example microscopy).

EXPECTED VALUES

Cryptosporidium spp. infections occur mostly in children younger than 2 years of age, with a peak in children younger than 2 years of age. In immuno-deficient humans, especially individuals with HIV/AIDS, cryptosporidiosis can be associated with chronic, potentially life-threatening diarrhea.

In developing countries, about 200 million people have symptomatic giardiasis, with some 500000 new cases reported each year. *Cryptosporidium* accounts for up to 20% of all cases of childhood diarrhoea, whereas invasive amoebic infection by *E. histolytica* affects 50 million people worldwide each year, resulting in 40000-100000 deaths annually.

Among the main infectious diarrheagenic pathogens, *Cryptosporidium spp.* results in the most deaths among children <5 years of age.

PERFORMANCE CHARACTERISTICS

Clinical sensitivity and specificity

Several evaluations, with faecal samples, was performed using **Vitassay Crypto+Giardia+Entamoeba** and these results were conpared using a microscopy technique, PCR for Crypto and Giardia strips and qPCR test (VIASURE *Entamoeba histolytica* Real Time Detection Kit and VIASURE *Entamoeba dispar* Real Time Detection Kit, CerTest) for Entamoeba strip.

Results	were	as	follows:

		Microscopy technique/PCR		
		Positive	Negative	Total
Vitassay	Positive	25	0	25
Crypto+Giardia +Entamoeba	Negative	0	229	229
Cryptosporidium	Total	25	229	254

Vitassay Crypto+Giardia+Entamoeba (Cryptosporidium) vs Microscopy technique/PCR						
	Mean Value 95% confidence interval					
Sensitivity	>99%	86.3-100%				
Specificity	>99% 98.4-100%					
PPV	>99% 86.3-100%					
NPV	>99%	98.4-100%				

		Microscopy technique/PCR		
		Positive Negative Total		Total
Vitassay Crypto+Giardia	Positive	61	0	61
+Entamoeba	Negative	2	191	193
Giardia	Total	63	191	254

Vitassay Crypto+Giardia+Entamoeba (Giardia) vs Microscopy technique/PCR						
Mean Value 95% confidence interval						
Sensitivity	97% 89.0-99.6%					
Specificity	98.1-100%					
PPV	PPV >99% 94.1-100%					
NPV	99%	96.3-99.9%				

		VIASURE <i>Entamoeba histolytica</i> Real Time Detection Kit		
		Positive Negative Total		
Vitassay	Positive	4	1	5
Crypto+Giardia +Entamoeba	Negative	1	107	108
Entamoeba	Total	5	108	113

Vitassay Crypto+Giardia+Entamoeba (Entamoeba) vs VIASURE Entamoeba histolytica Real Time Detection Kit					
Mean Value E. histolytica 95% confidence interval					
Sensitivity	80%	28.4-99.5%			
Specificity	99%	94.9-100%			
PPV	80% 28.4-99.5%				
NPV	97%	94.9-100%			

		VIASURE Entamoeba dispar Real Time Detection Kit		
		Positive Negative Total		
Vitassay	Positive	15	1	16
Crypto+Giardia	Negative	6	107	113
Entamoeba	Total	21	108	129

Vitassay Crypto+Giardia+Entamoeba (Entamoeba) vs VIASURE Entamoeba dispar Real Time Detection Kit						
Mean Value E. dispar 95% confidence interval						
Sensitivity	71% 47.8-88.7%					
Specificity	99% 94.9-100%					
PPV	PPV 94% 69.8-99.8%					
NPV	95%	88.8-98.0%				

		VIASURE Entamoeba histolytica Real Time Detection Kit and VIASURE Entamoeba dispar Real Time Detection Kit		
		Positive	Negative	Total
Vitassay	Positive	19	2	21
Crypto+Giardia Negative		7	214	221
+Entamoeba Entamoeba	Total	26	216	242

Vitassay Crypto+Giardia+Entamoeba (Entamoeba) vs VIASURE
Entamoeba histolytica Real Time Detection Kit and VIASURE
Entamoeba dispar Real Time Detection Kit

	Mean Value E. histolytica+ E. dispar	95% confidence interval
Sensitivity	73%	52.2-88.4%
Specificity	99%	96.7-100%
PPV	90%	69.6-98.8%
NPV	97%	93.6-98.7%

The results showed that **Vitassay Crypto+Giardia+Entamoeba** has a high sensitivity and specificity to detect *Cryptosporidium*, *Giardia y Entamoeba (histolytica o dispar)*.

Cross reactivity

No cross reactivity was detected against other gastrointestinal pathogens that are occasionally present in feces.

	Campylobacter jejuni	Giardia lamblia (Strip A and C)	Salmonella typhimurium
	Campylobacter coli	Helicobacter pylori	Shigella boydii
L	Clostridium difficile	Listeria monocytogenes	Shigella dysenteriae
	Cryptosporidium parvum (Strip B and C)	Salmonella enteritidis	Shigella flexneri
	Escherichia coli 0157:H7	Salmonella paratyphi	Shigella sonnei
A	Entamoeba hystolitica (Strip A and B)	Salmonella typhi	Staphylococcus aureus



REFERENCES

1. TAHMINA AHMED; HAMIDA KHANUM; MUHAMMED SALAH UDDIN; PRIYANKA BARUA; TUHINUR ARJU; MAMUN KABIR; RASHIDUL HAQUE. "*Entamoeba Histolytica, Giardia Lamblia* and *Cryptosporidium* spp. infection in children in an urban slum area of Bangladesh". Bioresearch Communications, Vol. 02, issue 01, January 2016, pp. 175-181.

2. MARIA J. FLECHA; CYNTHIA M. BENAVIDES; GABRIEL TISSIANO; ABRAHAM TESFAMARIAM; JUAN CUADROS; AIDA DE LUCIO; BEGOÑA BAILO; LOURDES CANO; ISABEL FUENTES; DAVID CARMENA. "Detection and molecular characterization of Giardia duodenalis, Cryptosporidium spp. and Entamoeba spp. among patients with gastrointestinal symptoms in Gambo Hospital, Oromia region, southern Ethiopia". Tropical Medicine and International Health, vol. 20, n^o 9, September 2015, pp. 1213-1222.

3. MARIT G. TELLEVIK; SABRINA J. MOYO; BJORN BLOMBERG; TORUNN HJOLLO, SAMUEL y. MASELLE; NINA LANGELAND; KURT HANEVIK. "Prevalence of Cryptosporidium parvum/hominis, Entamoeba histolytica and Giardia lamblia among Young Children with and without Diarrhea in Dar es Salaam, Tanzania". PLoS Negl Trop Dis 9(10): e0004125, October 2015.

SYMBOLS FOR IVD COMPONENTS AND REAGENTS

IVD	in vitro diagnostic device	Ť	Keep dry
Ĩ	Consult instructions for use	X	Temperature limitation
Σ	Use by		Manufacturer
LOT	Batch code	Σ _n	Contains sufficient for <n> test</n>
DIL	Sample diluent	REF	Catalogue number









